ERGOLINE®

ERGOLINE Aktiv · ERGOLINE Passiv · ERGOLINE Carbon

CE

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Contents

1.	Introduction	28
2.	ERGOLINE seat shell and available accessories	28
3.	ERGOLINE standard equipment	29
3.1	ERGOLINE seat shell with customized upholstery	29
3.2	Using the seat shell	30
3.3	Restraints in the seat shell	31
4.	Arm positioning	34
4.1	Arm rests, height-adjustable (standard arm upholstery)	34
4.2	Arm rests, incrementally height- and angle-adjustable (standard arm upholstery)	34
4.3	Arm cushion, trough-shaped	35
4.4	Lower arm restraint	35
4.5	Arm rests ALULINE	35
5.	Head positioning	36
5.1	Head rest bracket, angled	36
5.2	Head rest bracket, straight	37
5.3	Head rest bracket, continuous angle adjustment	37
5.4	Head rest cushion	37
5.5	Headrest systems ALULINE	38
6.	Leg positioning	39
6.1	Abduction pommel (spreading wedge)	39
6.2	Abduction belt with push-in buckle	40
6.3	Abduction belt with deflection roller	40
7.	Lateral chest restraint	41
7.1	Chest belt with flap	41
7.2	Chest belt with deflection roller	41
7.3	Reclining supports	41
7.4	Shoulder bands	42
7.5	Lateral side supports with swing-away unit	43
7.6	Lateral side supports without swing-away unit	43
8.	Back angle adjustment (optional)	43
9.	Cleaning	44
10.	Safety instructions	44
11.	Reuse and disposal	45
12.	Declaration of conformity	46
13.	Service plan	47

1. Introduction

Congratulations on your purchase of a seat shell by interco!

This seat shell is a high-quality product. It has been manufactured and inspected using the latest technology in cooperation with your medical supplies dealer. We have provided a description of the individual components in these operating guidelines in order to make it easier for you to handle the product. Please be aware that these guidelines may include descriptions of components that are not included in your product. It is important to adjust the product to meet the needs of each individual patient. The list of accessories and their corresponding descriptions is long in order to meet these many demands.

Please read the sections concerning your special model carefully and thoroughly and keep these operating guidelines in a safe place so that you can refer to them later, as needed. If you should have any questions about your seat shell, please ask us or your qualified medical supplies dealer.

2. ERGOLINE seat shell and available accessories

The ERGOLINE seat shell is a seating aid suitable for insertion in a normal standard folding wheel chair. It is used to provide the patient with proper support and positioning while seated in the wheel chair and can be adjusted to meet the individual needs of each patient. The low overall height will not limit the patient's mobility and the wheel chair and all of its features will remain fully functional.

ERGOLINE essentially comprises two components, including:

- ▶ ERGOLINE adapted seat unit
- ▶ ERGOLINE adapted back unit

These two components are connected by means of hinges. However, the seat and back can also be used individually. In addition, a variety of additional components, restraints, and aids can also be mounted to the seat shell. These individual components are described in the section dealing with the restraint of the corresponding part of the patient's body. Please see also the Table of Contents.

ERGOLINE is available in two versions:

- ▶ ERGOLINE Aktiv (active seating system)
- ▶ ERGOLINE Passiv (passive seating system)

Both versions are available in aluminium construction. ERGOLINE Aktiv is also available in carbon lightweight construction (ERGOLINE Carbon).

3. ERGOLINE standard equipment

The ERGOLINE seat shell standard equipment includes several components described in this separate section. Please see the Table of Contents for an exact list. All of the other restraints and individual options are described in sections 4 through 7.

3.1 ERGOLINE seat shell with customized upholstery

The ERGOLINE seat shell is equipped and covered with customized upholstery. You can wash this cover. Please see section 9 (Cleaning) for washing instructions. The section below describes how to remove the cover.

3.1.1 Removing the cover

In order to be able to remove the cover, it is necessary to remove the hinges that connect the seat unit and back unit, as well as any other components that are bolted to the seat through the cover. If you are using the ERGOLINE seat or back unit only, then it is not necessary to remove the hinges. You will need a crosstip screwdriver in order to remove the hinges.

To do so, proceed as follows:

- ▶ Turn the seat shell until the fastening screws are visible on the hinges.
- ▶ Remove each screw by turning it counterclockwise. The seat and back units are now separated.
- ▶ Repeat these steps in reverse order to assemble the seat and back units.

To remove the covers, proceed as follows:

- ▶ Open the zippers on the seat and back units and pull off each cover.
- ▶ Please follow the instructions under section 9 (Cleaning) when washing the covers.
- ▶ Repeat these steps in reverse order to replace the covers.

3.1.2 Mounting the ERGOLINE seat shell to the wheel chair

The seat shell has been equipped with corresponding fastening straps for mounting it to the wheel chair. To mount it, proceed as follows:

- ▶ Fold the ERGOLINE seat shell together as far as it will go, ensuring that the seating and back surfaces are folded as tightly together as possible.

- ▶ Open the hook and loop fastener on the back side of the ERGOLINE seat shell. Secure the ERGOLINE seat shell to the seating surface on the wheel chair using the hook and loop fasteners.
- ▶ Secure the back of the ERGOLINE to the back frame on the wheel chair using the two belts.
- ▶ Place the ERGOLINE seat shell on the seating surface of the wheel chair, ensuring that the back edge of the shell is flush against the back of the wheel chair
- ▶ Pull the hook and loop fasteners on the seat shell around the back frame of the wheel chair and fasten them together, ensuring that they are as tight as possible.
- ▶ Then fold the back of the seat shell against the back frame and pull the straps around the back frame.
- ▶ Thread one end of the strap into the push-in buckle on the opposite strap and pull the straps on the back of the seat shell around the back frame of the wheel chair, ensuring that they are as tight as possible. Secure the straps using the push-in buckle. The push-in buckle must click into place with an audible “click” sound.
- ▶ Repeat these steps in reverse order to remove the seat shell from the wheel chair.

3.2 Using the seat shell

In order to place and secure the patient in the seat shell there are several basic rules to follow. It is not possible to include all of the possible variations and special cases in this brief section. If you have any doubts concerning how to handle the seat shell, please ask your medical supplies dealer or speak with your therapist. They are your best resource if you should experience any problems.

When placing the patient in the seat shell, please proceed carefully as follows:

- ▶ Do not place the patient in the seat shell unless it has been properly mounted to the wheel chair and is equipped with the necessary restraint options.
- ▶ Always check to ensure that the seat shell is securely connected to the wheel chair. Please see section 3.1.2 (Mounting the ERGOLINE seat shell to the wheel chair) to learn how to check this.
- ▶ Check to ensure that all of the brakes on the wheel chair have been engaged. Please read the corresponding section in the operating guidelines for your wheelchair to learn how to do so.
- ▶ You can then place the patient in the seat shell. Please check to ensure that all of the restraints (except for the pelvic harness) are out of the way

and that the patient is not sitting on them. In addition, check to ensure that no hard objects are in the seat shell that could cause discomfort to the patient.

- ▶ The patient's pelvis should be as far back in the seat shell as possible. This is the only way to ensure optimal restraint and support.
- ▶ Always work from the patient's pelvis outwards when placing additional restraints. Do not proceed with positioning and restraining the torso and extremities, where applicable, until the pelvis has been optimally restrained.

3.3 Restraints in the seat shell

The options for restraining the patient's body in the seat shell include:

- ▶ Chest harness
- ▶ Pelvic bands
- ▶ Lap belt
- ▶ Pelvic harness

Please see section 4 through 7 to learn about all of the other restraint and support options and how to operate them.

3.3.1 Chest harness

The chest harness is used to restrain the patient's entire upper body. The 2 sections below describe how to secure the chest harness.

Important: Always adjust the chest harness, working from the hard back of the shell outwards and using the 4 tighteners located there.

Restraining the patient in the seat shell

Preparation: Place the patient in the seat shell and lay the chest harness over the back edge of the shell.

To restrain the patient, proceed as follows:

- ▶ Lay the chest harness forward with each of the two shoulder straps on either side of the patient's head.
- ▶ The lower end of the chest harness is equipped with two tighteners for fastening the straps.
- ▶ Open the flap on one of the black strap tighteners.
- ▶ Take one of the straps that is sticking out between the seat cushion and the back cushion and insert it into the corresponding slot between the moving flap and the fixed part.
- ▶ Pull the strap as far as it will go.

- ▶ Hold the strap in this position.
- ▶ Then close the flap on the tightener until it lies flat against the fixed part.
- ▶ The strap is now secured.
- ▶ To open the lock, pull on flap on the tightener and pull out the strap.
- ▶ Repeat this procedure on the other side.
- ▶ Repeat these steps in reverse order to release the chest harness.

Adjusting the chest harness

Preparation: Place the patient in the seat shell and apply the chest harness.

Important: Always adjust the chest harness, working from the hard back of the shell outwards and using the 4 tighteners located there.

To adjust the chest harness, proceed as follows:

- ▶ Adjust the lower ends of the chest harness first.
- ▶ Open the lower tighteners on the hard back of the seat shell.
- ▶ To do so, simply pull hard on the free end of the strap and the tightener will open automatically.
- ▶ Pull the strap on one side to the desired position.
- ▶ Hold the strap in this position.
- ▶ Then close the flap on the tightener until it lies flat against the fixed part.
- ▶ Repeat this procedure to adjust the other side.
- ▶ Repeat these steps to adjust the upper ends of the chest harness, using the upper tighteners on the hard back of the seat shell. The procedure is identical for both the upper and lower ends of the chest harness.

3.3.2 Pelvic band

Preparation: Place the patient in the seat shell and connect the pelvic band firmly to the seat shell on one side.

- ▶ Lay the pelvic band over the patient's lap.
- ▶ Then thread the free end of the strap into the clamping buckle while simultaneously pressing the pelvic band with one hand against the patient's pelvis. Press on the moving part on the buckle using the thumb on the hand that you are using to press the pelvic band and then thread the free end of the strap into the buckle using your other hand.
- ▶ Pull on the strap until the pelvic band has reached its final position. You can verify this by checking to ensure that the band is positioned properly and pressure is evenly distributed along the patient's pelvis. The two large cushions on the pelvic band should be centered on the two sides of the patient's pelvis to distribute pressure evenly.

- ▶ To open the buckle, press on the moving part. This will release the strap and you can then pull it out. Please make sure to hold the patient in place in the seat shell when performing this step.

3.3.3 Lap belt with press buckle

Preparation: Place the patient in the seat shell.

- ▶ Lay the lap belt over the patient's lap with the inserter.
- ▶ Press the inserter into the buckle while simultaneously pressing with one hand against the patient's pelvis.
- ▶ Then pull on the free end of the strap until the lap belt has reached its final position. You can verify this by checking to ensure that the belt is positioned properly and pressure is evenly distributed along the patient's pelvis.
- ▶ To open the buckle, press hard on the red button in the middle of the buckle. Please make sure to hold the patient in place in the seat shell when performing this step.

Note: It is usually only necessary to adjust the belt once at the beginning. This adjustment will be maintained even after you open the buckle. However, you must check to ensure proper adjustment with each use.

3.3.4 Lap belt with deflection roller

Preparation: Place the patient in the seat shell.

- ▶ Lay the part of the lap belt with the deflection roller over the patient's lap.
- ▶ Then pull on the other end of the belt using the deflection roller.
- ▶ Then pull on the free end of the belt until the lap belt has reached its final position. You can verify this by checking to ensure that the belt is positioned properly and pressure is evenly distributed along the patient's pelvis.
- ▶ Secure it into position by connecting the free end of the belt with the hook and loop fastener provided for this purpose.
- ▶ To open the lap belt, release the hook and loop fastener.

Note: The belt must be adjusted with each use. Always check to ensure that the belt is positioned properly each time you use it.

3.3.5 Pelvic harness

Preparation: Before placing the patient in the seat shell, always check to ensure that the pelvic harness is sitting flat against the seating surface with no folds. Then place the patient in the seat shell as described in section 3.2. (Using the seat shell).

Then, proceed as follows:

- ▶ The pelvic harness is divided into two sections at the end toward the patient's knees. The harness is equipped with one belt with a push-in buckle for each leg.
- ▶ Lay one of the belts from one side to the other over the patient's thigh and use one hand to press the press-in buckle inserter into the press-in buckle. This receptacle is located at the corresponding edge of the seat shell (right or left side, depending on which leg you are restraining).
- ▶ Repeat this procedure on the other side.
- ▶ Check to ensure that the pelvic harness has been positioned properly. The belt straps should support the patient's pelvis without being too tight. The harness should not have any folds or the like (risk of pressure areas!)

4. Arm positioning

The components for positioning the arms are connected to the hard outer shell with universal brackets. Any adjustment involves changing the insertion depth of the square tube.

To do so, proceed as follows:

- ▶ Turn the black knurled knob counterclockwise until the square tube slides easily.
- ▶ Slide the square tube into the desired position.
- ▶ Then turn the black knurled knob clockwise to resecure the square tube in place.
- ▶ Repeat this procedure on the other side.

4.1 Arm rests, height-adjustable (standard arm upholstery)

The height-adjustable arm rests are equipped with a standard cushion and secured to the seat shell with universal brackets. Please read section 4 (Arm positioning) to learn about the steps required to adjust the arm rest height.

4.2 Arm rests, incrementally height- and angle-adjustable (standard arm upholstery)

The height- and angle-adjustable arm rests are equipped with a standard cushion and secured to the seat shell with universal brackets. Please read section 4 (Arm positioning) to learn about the steps required to adjust the arm rest height.

To adjust the angle, proceed as follows:

- ▶ Turn the black lever counterclockwise to open the clamping unit on one side of the arm rest bracket.
- ▶ The angle of the arm rest to the seat shell can then be adjusted.
- ▶ Hold the arm rest in the desired position with one hand and secure it in place by tightening the black clamping lever with the other hand.
- ▶ This will maintain the arm rest in position.
- ▶ Repeat this procedure on the other side.

4.3 Arm cushion, trough-shaped

Trough-shaped cushions are used to replace standard arm cushions and have a different shape that provides a sort of cushioned trough for the patient's arm. Read section 4.2 (Arm rests, incrementally height- and angle-adjustable (standard arm upholstery)) to learn how to adjust the height and angle.

4.4 Lower arm restraint

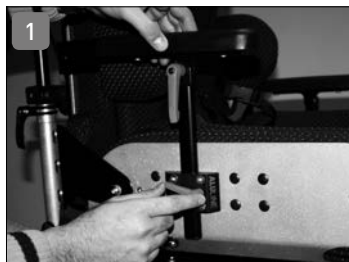
Lower arm restraints are used with trough-shaped arm cushions.

To restrain the lower arms, proceed as follows:

- ▶ Fold the restraint straps away from the surface of the rest.
- ▶ Place the patient's lower arm on the trough-shaped arm cushion.
- ▶ If necessary, adjust the height and angle of the arm rest, as described in section 4.2 (Arm rests, incrementally height- and angle-adjustable (standard arm upholstery)).
- ▶ Then use the hook and loop fasteners on the restraint straps to restrain the arm without constricting it.
- ▶ Repeat this final step with the second restraint strap.
- ▶ Repeat this procedure on the other side, as needed.


4.5 Arm rests ALULINE

Alternatively to standard arm positioning systems ERGOLINE can be equipped with different arm rests from the series ALULINE. Fix the arm rest by loosening the clamping lever of the arm rest receptacle on the right and left side of the seat shell, insert the arm rest holder and tighten the lever again. Adjustments are always done by changing the depth of the insertion in the square pipe (Photo 1).

**To adjust the angle of the arm rest
proceed as follows:**

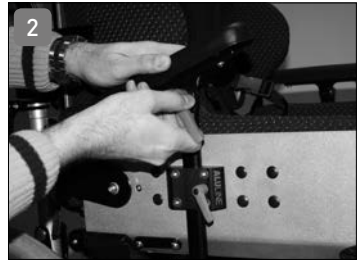- ▶ Turn the fixing lever anticlockwise to open the fixture (Photo 2).
- ▶ The arm rest angle now can be adjusted.
- ▶ Change the square pipe to the desired position and turn the black fixing lever clockwise to fix the square pipe.
- ▶ The position of the arm rest will now stay the same.
- ▶ Repeat the same procedure on the other side.

5. Head positioning

The components for positioning the head are connected to the hard outer shell with universal brackets. Any adjustment involves changing the insertion depth of the square tube.

To do so, proceed as follows:

- ▶ Turn the black knurled knob counterclockwise until the square tube slides easily.
- ▶ Slide the square tube into the desired position.
- ▶ Then turn the black knurled knob clockwise to resecure the square tube in place.
- ▶ Repeat this procedure on the other side.

5.1 Head rest bracket, angled

The angled head rest bracket can be used to adjust the head rest at two levels, height and depth.

To adjust the height, proceed as follows:

- ▶ The universal bracket for the head rest bracket is located on the back side of the hard outer shell. To adjust it, proceed as described under section 5. "Head positioning".

To adjust the depth, proceed as follows:

- ▶ Turn the wing screw counterclockwise to release the clamping device on the top cross bar.
- ▶ Slide the head rest bracket tube into the desired position in the cross bar.

- ▶ Secure it into position by turning the wing screw clockwise until it is no longer possible to slide the head rest bracket tube.

5.2 Head rest bracket, straight

The angled head rest bracket can be used to adjust the head rest at two levels, height and depth. The procedure is identical to the one described under section 5.1 (Head rest bracket, angled).

5.3 Head rest bracket, continuous angle adjustment

The head rest bracket with continuous angle adjustment can be used to adjust the head rest at two levels, height and depth.

To adjust the height, proceed as follows:

- ▶ The universal bracket for the head rest bracket is located on the back side of the hard outer shell. To adjust it, proceed as described under section 5 (Head positioning).

To adjust the depth, proceed as follows:

- ▶ Turn the black clamping lever counterclockwise to open the clamping unit on the arm rest bracket.
- ▶ The angle of the head rest bracket to the seat shell can then be adjusted.
- ▶ Hold the head rest in the desired position with one hand and secure it in place by tightening the black clamping lever with the other hand.
- ▶ This will maintain the head rest in position.
- ▶ However, it is necessary to readjust the head rest cushion.
- ▶ Turn the head rest at its fixation point until the patient's head is positioned comfortably on the head rest cushion.

5.4 Head rest cushion

There are three different head rests available for the ERGOLINE seat shell that are secured using the bracket systems described above. The cushions available for these head rests include:

- ▶ Standard
- ▶ Shell-shaped
- ▶ Half-roll

The head rest cushions with hook and loop fasteners and cushions for the back of the head are not mounted using head rest brackets, but are attached to the back cushion with hook and loop fasteners.

5.4.1 Head rest cushions: Standard, shell-shaped, half-roll

These head rest cushions are bolted to the head rest bracket systems, including angled, straight or continuous angle adjustment. Please perform any basic adjustments to the head rest using the bracket systems.

5.4.2 Head rest with Velcro attachment, cushion for back of the head

The head rests with Velcro attachment or the cushions for back of the head are secured to the back cushion with hook and loop fasteners. In such cases, the back of the seat shell is extended to a height above the patient's head. These two head rest versions are only able to be mounted with this type of seat shell configuration.

To mount one of these two cushions, proceed as follows:

- ▶ Lay the back side of the head cushion (with the hook end of the fastener) on the back cushion (with the loop end of the fastener) and press gently. The cushion is now secured.
- ▶ To remove the cushion, simply pull hard on the head cushion until it releases from the back cushion.

5.5 Headrest systems ALULINE

Alternatively to standard headrest systems ERGOLINE can be equipped with different head rest systems from the series ALULINE (straight or shell shaped). Both are adjustable in height as well as depth. Fix the head rest by loosening the clamping lever of the head rest receptacle on the upper part of the back rest, insert the head rest holder and tighten the lever again. Adjustments are always done by changing the depth of the insertion in the square pipe (Photo 3).

To adjust the depth proceed as follows:

- ▶ Turn the fixing lever anticlockwise until the square pipe can be moved (Photo 4).
- ▶ Change the square pipe to the desired position.
- ▶ Turn the black fixing lever clockwise to fix the square pipe.

6. Leg positioning

Various cushions and straps are used to position the patient's legs.

6.1 Abduction pommel (spreading wedge)

Three different abduction pommels are available for ERGOLINE: integrated, adjustable and swing-type.

6.1.1 Abduction pommel integrated

The integrated abduction pommel was affixed during manufacturing of the seat shell. Adjustments can not be made after that. Should adjustments become necessary, please contact your specialist dealer.

6.1.2 Abduction pommel adjustable

The adjustable abduction pommel is affixed at the bottom of the outer side of the hard seat shell. An adjustment is possible. The position of the pommel toward the inner seat can be adjusted. It should be aimed that both knees are centered toward the abduction pommel side.

To adjust the abduction pommel proceed as follows:

- ▶ Loosen the clamp by turning the wing screw anticlockwise. The wing screw is placed under the seat at the mounted holder of the abduction pommel.
- ▶ Push the abduction pommel into the correct position.
- ▶ Fix the position by turning the wing screw clockwise until the abduction pommel can no longer be moved.

6.1.3 Abduction pommel swing-type

The swing-type pommel is also affixed at the bottom of the outer side of the hard seat shell. For adjustments proceed like with the adjustable abduction pommel (Chapter 7.6.2).

To swing the abduction pommel forward proceed as follows:

- ▶ Loosen the catch by pressing the red button underneath the abduction pommel (Photo 5).
- ▶ Now you can swing back the pommel.
- ▶ To bring it back into its original position, move it back until it audibly locks.

6.2 Abduction belt with push-in buckle

An abduction belt is used to restrain the thigh to the thigh cushion on the seat shell.

To apply the abduction belt, proceed as follows:

- ▶ One part of the abduction belt originates in the fold between the seat cushion and the thigh cushion; it is equipped with a black press-in buckle inserter.
- ▶ The other part of the abduction belt is located on the outside of the thigh cushion bolt and is equipped with a push-in buckle receptacle.
- ▶ Lay the part of the abduction belt with the push-in buckle inserter over the patient's thigh and check to ensure that it is lying flat and not twisted.
- ▶ Insert the push-in buckle inserter into the push-in buckle receptacle until it audibly clicks into place.
- ▶ If it is necessary to readjust the belt length, proceed as follows: Pull the belt tighter by pulling the free end of the belt strap away from the plastic buckle until the desired tension is achieved. Loosen the belt by turning the belt buckle until it is perpendicular to the belt strap and then pull it away from the seat shell.
- ▶ Adjust the belt strap length to ensure that it is secure but not constricting.

6.3 Abduction belt with deflection roller

An abduction belt is used to restrain the thigh to the thigh cushion on the seat shell.

To apply the abduction belt, proceed as follows:

- ▶ One part of the abduction belt originates in the fold between the seat cushion and the thigh cushion; it is equipped with a deflection roller.
- ▶ The other part of the abduction belt is located on the outside of the thigh cushion bolt and is equipped with a hook and loop fastener.
- ▶ Lay the part of the abduction belt with the deflection roller over the patient's thigh and check to ensure that it is lying flat and not twisted.
- ▶ Then insert the other end of the belt (that is not fastened together) through the deflection roller.
- ▶ Pull the free end until the desired adjustment has been achieved.
- ▶ Secure the hook and loop fastener; the adjustment is now secure.
- ▶ Adjust the belt strap length to ensure that it is secure but not constricting.

7. Lateral chest restraint

There are various options available for the ERGOLINE. Each seat shell must have a basic restraint system. Please see section 3.3 (Restraints in the seat shell) for more information. The sections below describe additional restraints.

7.1 Chest belt with flap

The chest belt with flap has been mounted in the proper position during adjustment of the seat shell. It is possible to perform minor adjustments to the length using the hook and loop fastener.

To open the belt, proceed as follows:

- ▶ Pull upwards on the front flap.
- ▶ Then pull the overlapping ends of the chest belt apart.
- ▶ You can then lay the two ends of the chest belt aside and remove the patient from the seat shell, as necessary.
- ▶ Repeat these steps in reverse order when applying the belt to secure the patient in the seat shell.
- ▶ When applying the belt, check to ensure that it is not constricting the patient; this is especially important to ensure that the patient's breathing is not adversely affected.

7.2 Chest belt with deflection roller

The chest belt with deflection roller has been mounted in the proper position during adjustment of the seat shell. It is possible to perform minor adjustments to the length using the hook and loop fastener.

To open the belt, proceed as follows:

- ▶ Pull the overlapping ends of the chest belt apart.
- ▶ Pull the loose end of the chest belt through the metal loop.
- ▶ You can then lay the two ends of the chest belt aside and remove the patient from the seat shell, as necessary.
- ▶ Repeat these steps in reverse order when applying the belt to secure the patient in the seat shell.

7.3 Reclining supports

The reclining supports have been built into the postural side supports during production of the seat shell. You can adjust them by adjusting the distance between the two front "wings", i.e. the length of the connecting belt strap.

Preparation: Place the patient in the seat shell.

To secure the reclining supports, proceed as follows:

- ▶ Fold the two reclining supports forward and lay them on the patient's sternum.
- ▶ Open the flap on the black strap tightener.
- ▶ Insert the belt strap into the slot between the moving flap and the fixed part.
- ▶ Pull the belt strap until the cushion lies loosely on the patient's sternum while the patient is seated upright in the seat shell.
- ▶ Hold the strap and reclining supports in this position.
- ▶ Then close the flap on the tightener until it lies flat against the fixed part.
- ▶ The strap is now secured.
- ▶ To open the lock, pull on flap on the tightener and pull out the strap. You can then fold the two "wings" out of the way and remove the patient from the seat shell.

7.4 Shoulder bands

The shoulder bands hold the shoulders against the back of the shell. The shoulder bands have been set up during the basic adjustments performed by your medical supplies dealer. If it is necessary to perform any readjustments, please contact your medical supplies dealer.

To apply and remove the shoulder bands, proceed as follows:

Preparation: Place the patient in the seat shell.

- ▶ Release the black lever located on the connection between the shoulder cushion bar and the actual support structure by turning it counterclockwise.
- ▶ It is then possible to slide and turn the shoulder band.
- ▶ Perform the following movements by taking the upholstered part of the shoulder band in one hand:
 - ▶ Pull the shoulder band away from the patient's body somewhat
 - ▶ Turn the shoulder band 90° until it is horizontal with the edge of the seat.
 - ▶ Slide the shoulder band back toward the back of the shell.
 - ▶ Repeat this procedure on the other side.
- ▶ The patient is then no longer restrained by the shoulder bands.
- ▶ Repeat these steps in reverse order to apply the shoulder bands. When applying the shoulder bands, check to ensure that pressure is evenly distributed on the shoulders and not excessive.

7.5 Lateral side supports with swing-away unit

The lateral side supports with swing-away unit have been built into the postural supports during production of the seat shell. It is no longer possible to perform any readjustments to the cushions. If it is necessary to perform any readjustments, please contact your medical supplies dealer.

To swing away the lateral side supports, proceed as follows:

- ▶ Take hold of the front of the cushion with one hand.
- ▶ Use your pointer finger on the other hand to press on the ribbed area of the silver release lever on the outside of the swing-away unit hinge.
- ▶ Then swing the postural side support away to the side (while still pressing the release lever!).
- ▶ To return the lateral side supports back to their starting position, simply swing them back into place until you hear an audible “click” sound.

7.6 Lateral side supports without swing-away unit

The lateral side supports without swing-away unit have been built into the side elements of the ERGOLINE back unit during production of the seat shell. It is not possible to perform any readjustments. If it is necessary to perform any readjustments, please contact your medical supplies dealer.

8. Back angle adjustment (optional)

Your ERGOLINE has been equipped with a back angle adjustment option. There are several rules to follow when using this option in order to ensure the safe use of your ERGOLINE.

The information below relates to the mounting of the ERGOLINE on the SIMPLY undercarriage manufactured by interco; the applicability of this information must be checked when using the ERGOLINE with any other undercarriage.

Version with incremental adjustment plates for back angle adjustment:

1. The incremental adjustment plate on the ERGOLINE back angle adjustment system is limited to the next-to-last adjustment level by the maximum inclination of the SIMPLY push bar. However, it should be mentioned here for your information, that the third-to-last adjustment level enables the back unit to be reclined to approx. 120°, which is usually sufficient. If you desire a greater angle, then adjust the seat angle setting on the SIMPLY undercarriage in order to maintain the lowest possible center of gravity for the product combination.

2. When using the back angle adjustment the anti-tipping wheels should be set to maximum width; otherwise the anti-tipping function cannot be guaranteed.
3. The wheelbase extension for the rear axle of the SIMPLY must be adjusted to the maximum wheelbase setting.
4. Never leave the patient in the seat shell while performing adjustments on the ERGOLINE or the SIMPLY.

Version with continuous back angle adjustment:

1. Continuous back angle adjustment enables a maximum angle of approx. 120°.
2. The same precautions as those described under items 1-4 (Version with incremental adjustment plates) must be taken when setting up the seat shell.

Important: If you use your ERGOLINE on any undercarriage other than the SIMPLY, we hereby expressly inform you that you do so at your own risk. As a result, we urgently recommend that you check for negative impacts on anti-tipping safety, shifts in the center of gravity, lateral stability, improperly functioning Bowden cables and brakes, etc.; consult your qualified dealer or interco in the case of any doubts.

9. Cleaning

- ▶ All of the fabrics for our covers are washable at 30 °C. Please see section 3.1.1 (Removing the cover) to learn how to remove the cover.
- ▶ Use a detergent for delicate fabrics.
- ▶ Never dry the covers in the dryer!
- ▶ If you should wish to wash the seat shell cover frequently, then it is advisable to have an auxiliary cover to put on the seat shell while washing the other cover. Ask about this at your medical supplies store.
- ▶ Please clean additional parts with a mild household soap solution.

10. Safety instructions

Please observe the following safety instructions:

In principle, the ERGOLINE seat shell has not been designed for use with an undercarriage manufactured by interco GmbH or with an undercarriage by another manufacture for the transport of patients while seated in the seat shell - with or without restraints - in private cars or vehicles for transporting disabled persons. The manufacturer shall assume no liability in the case of any personal damages occurring during such transport.

1. The seat shell is only approved for its intended use. It is used for positioning and restraining persons with corresponding physical characteristics.

In particular, the intended use does NOT cover the following:

- ▶ Use of the ERGOLINE as a seat by other persons,
 - ▶ Use of the ERGOLINE as a car seat,
 - ▶ Use of the ERGOLINE to stabilize the patient during performance of personal hygiene,
 - ▶ Use of the ERGOLINE following disassembly/modification of individual components.
2. The ERGOLINE seat shell is designed for insertion in standard folding wheel chairs. The restraints and strap systems have been designed for this purpose. As a result, other uses are not covered by the intended use.
 3. Always check to ensure that the seat shell is securely connected to the wheel chair.
 4. Do not mount any accessories that have not been tested and approved by the manufacturer.

11. Reuse and disposal

Reuse

ERGOLINE is suitable for reuse. Before it is passed on, all the necessary cleaning and maintenance activities must be performed. When it is handed over, make sure it is accompanied by all the documentation necessary for use (including these instructions for use).

Disposal

When ERGOLINE has reached the end of its service life, make inquiries with your local waste management company about the regulations concerning disposal and recycling.

12. Declaration of conformity

EU-Declaration of Conformity for Medical Devices

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Address of the manufacturer: Im Auel 50
53783 Eitorf GERMANY

We hereby declare that the device

Article description: ERGOLINE Aktiv/Passiv seat shell system
Version: Aluminium or carbon lightweight construction

conforms to the pertinent provisions of the directive listed below:

- ▶ Medical Device Directive 93/42/EEC, Annex I and VIII

Eitorf, 24 November 2000

Michael Markwald
CEO

13. Service plan

Important: Inspections must take place every 6 months!

Pos.	Area	Check	ok	Damage / deformation	Exchange / replacement
1	Function check	Back adjustment	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Fixing unit	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Arm rest			
		Head support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Abduction wedge	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Therapy table	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
2	Screw connections	Screws tightened	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Rust spots	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3	Appearance (visual inspection)	Paintwork	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Flock coating	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Cleanliness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Fabric seams (position / appearance)	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Type plate	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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Status as per March 2018
Subject to technical alterations
without prior notification

Zertifiziert nach
Certified as per
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 357-1
DIN EN ISO 13485:2012
Reg.-Nr. 70 105 357-1
TÜV Hessen

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© interco GmbH, Eitorf, 2018