MINY®
MINY®

Sitzschalenuntergestell · Seat unit frame

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E-Mail*: _____

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Contents

1.	Introduction	30
2.	Rating plate	30
3.	Miscellaneous	31
4.	Intended use/Field of application	31
5.	Features/options/adjustments	32
6.	Technical Data	36
7.	Operation and Assembly	37
7.1	Attaching the seat shell	37
7.2	Adjusting the inclination of the seat shell (tilt-in-space mechanism)	38
7.3	Adjustment options for the push bar	39
7.4	Folding mechanism for transporting the MINY	41
7.5	Unfolding the MINY	42
7.6	Drum brake-enabled back wheels	42
8.	Maintenance and care	43
8.1	Checking the tyre pressure	43
8.2	Adjusting the parking brake	44
8.3	Cleaning	45
8.4	Service Intervals	45
8.5	Service life	45
9.	Transport/Transportation lock/Stair climbing	46
10.	Safety instructions	46
11.	Possible disturbances	48
12.	Reuse and disposal	48
13.	Declaration of Conformity	49
14.	Warranty	50
15.	Service manual	51

1. Introduction

Congratulations, you have chosen the MINY seat unit frame by interco GmbH.

In order to ease the handling of the product for you, we described the application of the different components in the following user manual. Please be appreciative of the fact that we also describes parts that might not be found in on your product. Most important is the individual fitting to the patient's needs. In order to satisfy most claims, the list of described items is a little more comprehensive. Please read the parts that apply to your specific model carefully and keep the user manual for later consulting. For further questions please contact us or your specialised dealer.

MINY is the street undercarriage for the interco seat unit range and offers a variety of adjustment possibilities. The inclination of the seat unit can be continuously adjusted via gas spring. Pneumatic tyres and drum brakes supplied as standard ensure a high degree of safety and comfort. At the front, MINY features swivel wheels so that it is extremely manoeuvrable and particularly easy to push. MINY can be folded up easily and quickly for transportation and it fits into any car boot.

The MINY street undercarriage is suitable for all interco seat units featuring a trapezoidal adaptor. MINY has been especially designed for outdoor use. Due to the pivoting front wheels the device is also easy and convenient to use indoors. For safety reasons, driving with grip tires should be avoided when knee brakes (optional) are missing.

2. Rating plate

The rating plate (Fig. 1) is located on the rear connecting tube of the base frame. The rating plate provides the following information:

- ▶ Product name
- ▶ Article number
- ▶ Unique serial number (to be quoted in the event of a complaint or when asking questions about the product)

- ▶ Size of the rear wheels
- ▶ Seat width size
- ▶ Maximum user load
- ▶ CE marking

3. Miscellaneous

The product has been tested in a temperature range of -20 °C to +60 °C (-4 °F to +140 °F). Colour differences between individual parts cannot always be avoided. Unfortunately, any claims referring to this fact cannot be admitted. Please understand that improper handling or differences in the fabric colours, as described above, are no basis for a justified claim. A regular maintenance of all metal parts of the wheelchair, in particular the chrome parts, is necessary to preserve its high-grade appearance. Conventional cleaning products are sufficient for this purpose. Small repair sets for damages to the frame's paintwork are available at a specialised dealer. The extremities of the axle have to be cleaned and slightly lubricated about every fourth week.

4. Intended use / Field of application

When supplying children with seat units and modular seating systems, undercarriages for indoor and/or outdoor use are compulsory. They increase the mobility and make the transport of the insured person possible. Therefore, the purpose of all undercarriages – in particular MINY – always is the reception of a seat unit.

Indication in conjunction with a seat shell:

- ▶ Infantile cerebral palsy
- ▶ Myelodysplastic syndrome
- ▶ Muscular dystrophy
- ▶ Various clinical pictures with scoliotic spinal deformities

Contraindication:

- ▶ none known

5. Features / options / adjustments

Modules (Fig. 2)


- ▶ Push bar (1)
- ▶ Back (2)
- ▶ Trapezoidal adapter unit (3)
- ▶ Rear wheels (4)
- ▶ Pivoting front wheels (5)
- ▶ Undercarriage frame (6)
- ▶ Leg rest system (7)

The basic configuration includes:

- ▶ Operating Guidelines
- ▶ aluminum base frame, slightly abducted version
- ▶ adapter receptacle, trapezoid-shaped for Interco seat shells, depth-adjustable - tilt-in-space mechanism using gas-pressure springs, 300 N
- ▶ back angle adjustment up to 30° with perforated plate
- ▶ tilt-in-space mechanism using gas-pressure springs up to max. 35°
- ▶ push bar with incremental adjustment, height- and angle-adjustable
- ▶ Tilting mechanism 2 in 1 (tilting protection and support)
- ▶ 12", 16", 22" or 24" rear wheels with quick-release axle, drum brake and aluminum grip rings
- ▶ pneumatic tires or solid tires
- ▶ standard front wheels in various sizes
- ▶ adjustable seat height and seat depth
- ▶ individual color selection from the entire RAL range


MINY additional options include:

- ▶ Height-adjustable ALULINE leg rest system with incrementally adjustable holder and anti-slip overlay
- ▶ Center leg rest system with incrementally adjustable holder, foot plate foldable up and back, and anti-slip overlay
- ▶ drum brake wheels with parking brake handle
- ▶ various spoke protectors
- ▶ knee lever brake system
- ▶ clothing guard / wheel protectors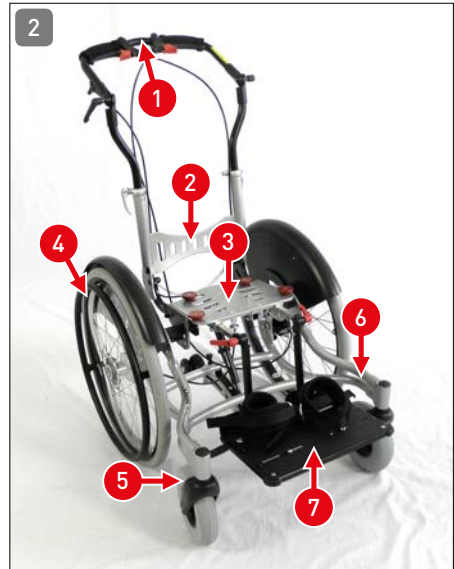

The mounted adapter receptacle, trapezoid-shaped is standard on all undercarriages by Interco GmbH. This makes it possible to mount any Interco seat shell with a trapezoidal adapter on various undercarriages by Interco GmbH.

The adapter receptacle, trapezoid-shaped mounted on the undercarriage essentially consists of 3 functional groups (Fig. 3):

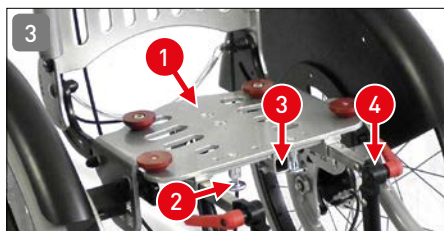- ▶ Mounting plate with the guide bushings compatible with the seat shell interface (1)
- ▶ Spring-loaded catch mechanism for securing the seat shell to be mounted and keeping it in place (2)
- ▶ Receptacle unit (centered) for mounting and attaching the center leg rest system with incrementally adjustable holder (3)
- ▶ Side mounting units for assembling and attaching the ALULINE leg rest system with incrementally adjustable holder (4)

Note: Please make sure that the interface system has been adjusted correctly in each individual case. This can be determined by ensuring that the seat shell interface is flush against the front edge on the adapter receptacle, trapezoid-shaped, and the spring-loaded catch mechanism snaps completely into place!

Centered leg rest system:

The centered leg rest is factory-mounted with your order. Subsequent changes are possible for individual customization.

Adjustment options:

The centered leg rest system has various adjustment options for optimal customization:

a) Vertical inclination adjustment range

In order to adjust the centered leg rest (Fig. 4), proceed as follows:

- ▶ Activate the parking brakes
- ▶ Loosen the screw connections on the centered leg rest by turning the clamp levers

counter-clockwise until the top piece becomes loose.

- ▶ Move the centered foot rest into the desired angle.
- ▶ Hold the centered foot rest in the desired angle with one hand and secure it in place by tightening the corresponding clamp lever with the other hand.
- ▶ Deactivate the parking brake, as appropriate

b) Foldable back

The foot plate is folded up and back by hand.

c) Height and depth adjustment

Disengage the clamping lever (Fig. 6) on the clamping block of the foot rest and set the appropriate length of lower leg. To adjust the depth (thigh length) undo the screw connection (Fig. 7) below the seat adapter. Tighten all the screws again when adjustment has been completed.

Knee lever brake (air brake)

Sometimes it is necessary for a wheelchair to have an additional brake system to deal with any difficulties arising from diminished braking capacity, e.g. due to conditions on the ground (sloped surfaces) or in the case of self-propelled wheelchairs. Knee lever brakes are used in addition to parking brakes. These are factory-mounted and adjusted with your order. Adjustment of the knee lever brake should be checked frequently because of the different basic setting compared to the factory setting (e.g. tire pressure)!

Tilting mechanism and protection

Tilting mechanism 2 in 1 (tilting protection and support). Sliding and lockable with locking pins.

Pull out the spring-loaded catch mechanism and push in the tilt stop system in order to use it as a tilt assist device (Fig. 8). Make sure you hear the spring-loaded catch mechanism snap into place and check to make sure it is secure.

Note: In this position there is no protection against tilting!

To prevent tilting, the tilt stop system must have snapped into place in the extended position.

Note: The tilt assist function is restricted because it can move in again!

Use as a tilt assist device:

Pull back the push bars slightly and at the same time step on the tilt assist device. As a result you can tilt the undercarriage onto the rear wheels without any great effort. In this way you can easily overcome obstacles when traveling forward or backward.

Note: The user must be physically capable of holding the undercarriage together with the seat shell and occupant securely!

Tip: Always pay attention that the tilting protection wheels together with the two small wheels point towards the ground when a patient is sitting in the system.

6. Technical Data

	MINY SB 30	MINY SB 35	MINY SB 40
Weight, including wheels in the basic configuration (without leg rest)	approx. 16,5 kg	approx. 17,0 kg	approx. 18,0 kg
Frame width without attachments (measured in abduction)	44 cm	49 cm	54 cm
Seat width	30 cm	35 cm	40 cm
Seat height adapter min. - max.	23-30 cm	23-30 cm	23-30 cm
Seat height from foot rest min. -max.	25-39 cm	15-39 cm	7-39 cm
Overall width (with wheel camber)	50 cm (+ 5 cm)	55 cm (+ 5 cm)	60 cm (+ 5 cm)
Max. length without push bar (24" wheels)	75 cm (78 cm)	75 cm (78 cm)	75 cm (78 cm)
Height / length folded	52/79 cm	52/79 cm	52/79 cm
Height to top adapter plate min. - max.	46 - 48 cm	46 - 48 cm	46 - 48 cm
Height of push bar min. - max.	98 - 110 cm	98 - 110 cm	98 - 110 cm
Seat shell inclination angle	-5° to 30°	-5° to 30°	-5° to 30°
Leg rest holder to seat area (adapter plate) - incremental adjustment	0° to 90°	0° to 90°	0° to 90°
Gripping wheel diameter	48 cm (22"-wheels) · 53 cm (24"-wheels)		
Turning radius	approx. 92 cm (22"-wheels)		
Max. load	60 kg	60 kg	60 kg
Wheel camber	6°	6°	6°

Materials used

Component	Material	Corrosion protection
Frame	High-strength	Powder coated
Push bars	aluminum tubing	Powder coated
Adapter plate	Aluminum	Powder coated
Fasteners	Steel	Galvanized

Admissible tyre pressure

Wheel type	Tyre pressure (bar) maximum
Front swivel wheels	2,5
Rear wheels 12"	2,5
Rear wheels 16"	2,5
Rear wheels 22"	4,0
Rear wheels 24"	4,0

Important! With pneumatic tires, always change the tire if it has a worn profile, has developed cracks or is damaged.

Static stability

Static stability uphill*	max. 10°
Static stability downhill*	max. 10°
Static stability to the side*	max. 10°
Max. obstacle height*	10 cm

* Depends on the settings of the wheelchair and on the user's posture and physical abilities

Tools required for adjustment and assembly

All adjustments	Open-ended wrench (8 mm)
-----------------	--------------------------

7. Operation and Assembly


7.1 Attaching the seat shell

The seat shell is attached to the adapter receptacle, trapezoid-shaped (Fig. 9) on the undercarriage with the trapezoidal adapter and secured in place with the two spring-loaded catch mechanisms.

First, apply the parking brakes to prevent MINY from accidentally rolling away.

First, extend the spring-loaded catch mechanisms (Fig. 10) and turn them 90 degrees. Set the back end of the seat shell interface on the front side of the adapter receptacle, trapezoid-shaped; then slide the seat shell until it stops at the back and secure it with the spring-loaded catch mechanisms (rotate through 90 degrees so you hear it snap into place). Finally, check the seat shell and the seat shell locking system to ensure they are secure.

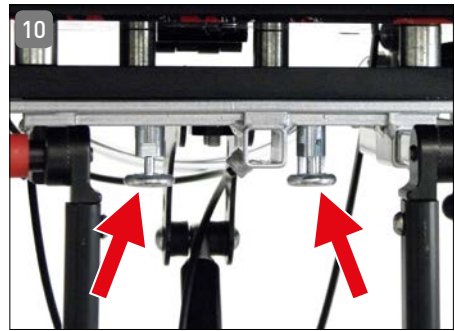Removing the seat shell

First, extend the spring-loaded catch mechanisms and turn them 90 degrees. Then slide the seat shell forward and remove it (Fig. 11).

7.2 Adjusting the inclination of the seat shell (tilt-in-space mechanism)

Due to inclination adjustment of the seat shell the stability of the MINY is only guaranteed if the tilt stop system is extended.

The MINY is equipped with a gas-pressure spring for the inclination of the seat shell. In order to adjust the inclination of the seat shell, proceed as follows:

- ▶ Activate the parking brakes
- ▶ Pull the gas-pressure spring release lever (Fig. 12) mounted to the right of the push bar frame.

- ▶ Adjust the inclination of the seat shell by pulling the push bar up or pressing it down while pulling on the gas-pressure spring release lever.
- ▶ Let go of the release lever to lock the seat shell into the adjusted inclination angle.
- ▶ Deactivate the parking brake, as appropriate

Important: Never pull the release lever without a firm grip on the push bar if a patient is seated in the wheelchair.


7.3 Adjustment options for the push bar

The push bar has various adjustment options. This makes it possible to achieve optimal customization to the person being pushed. The following adjustment options are possible:

7.3.1 Adjusting the angle of the top piece of the push bar (with incremental adjustment only)

In order to adjust the angle of the push bar (Fig. 13), proceed as follows:

- ▶ Activate the parking brakes
- ▶ Loosen the screw connections on the right and left ratchet joints on the push bar (1) by turning the clamp levers counter-clockwise until the top piece of the push bar becomes loose.
- ▶ Move the push bar into the desired angle (2).
- ▶ Hold the push bar in the desired position with one hand and secure it in place by tightening the corresponding clamp lever with the other hand.
- ▶ Secure the screw connection on the other side as well by tightening the clamp lever.
- ▶ When tightening the ratchet joints make sure that the teeth are engaging properly.
- ▶ Deactivate the parking brake, as appropriate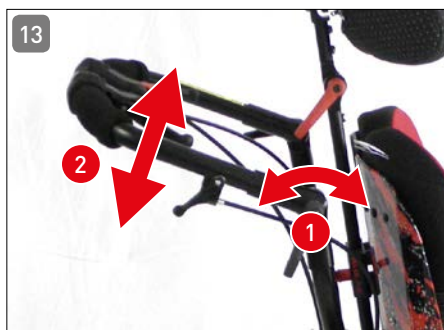


7.3.2 Adjusting the height of the push bar unit

The height of the push bar always needs to be adjusted when it is necessary to customize the MINY to the person being pushed or if the inclination of the seat shell has been changed.

In order to adjust the height of the push bar (Fig. 14), proceed as follows:

- ▶ First, loosen the quick-release fasteners (1) on the fork tubes by opening the two quick-release levers outwards.
- ▶ Then adjust the push bar to the desired height (2). To do this, pull or press the push bar to the desired height.
- ▶ Secure the push bar in position by tightening the quick-release connectors. To do this, tilt the lever on the quick-release fasteners back in.
- ▶ If the clamping pressure is not sufficient (push bar not secured in place), then turn the screw across from the quick-release fastener clamp lever (with lever in open position) counter-clockwise, until sufficient clamping pressure is achieved by closing the clamp lever. Repeat this step, as necessary.

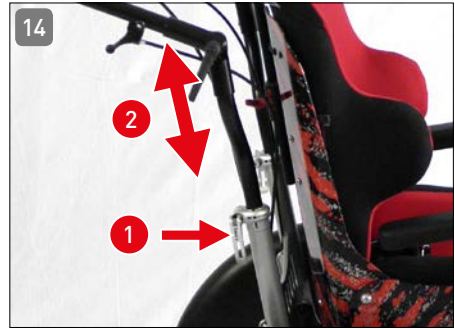7.3.3 Adjusting the inclination of the back unit with push bar

(Here, the back unit, including push bar, is folded back.)

In addition to the adjustment options described above, it is also possible to swivel the push bar backwards by up to 30° (in 5° increments) (Fig. 15).

This can become necessary, for example, if a seat shell is being used with a seat angle of more than 90°. This enables the gap between the back of the seat shell and the push bar to be as uniformly small as possible. This adjustment option is not intended for individual customization to the person being pushed.

In order to adjust the inclination of the push bar, proceed as follows:

- ▶ Loosen the spring-loaded locking system for the folding mechanism on the push bar by pressing down on the central cable system (1) connecting the two perforated plates to the bottom of the two back posts and simultaneously swiveling the push bar backwards (2).

- ▶ Then it is possible to shift the back unit including the push bar backwards (up to 30 degrees max.).

7.4 Folding mechanism for transporting the MINY

The MINY is equipped with a folding mechanism in order to make transport easier. The back unit including push bar is folded forward; it is also possible to remove the back wheels. This greatly condenses the size of the wheelchair for transport.

7.4.1 Folding up the MINY

In order to fold up the MINY, proceed as follows:

- ▶ Remove the seat shell (see Section 7.1)
- ▶ Loosen the height adjustment mechanism on the push bar (see Section 7.3.2)
- ▶ Loosen the screw connections on the push bar angle adjustment mechanisms on the right and left side of the push bar (see Section 7.3.1)
- ▶ Loosen the spring-loaded locking system for the folding mechanism on the push bar (Fig. 16) by pressing down on the central cable system connecting the two perforated plates to the bottom of the two back posts and simultaneously swiveling the push bar forward until the back together with the push bar comes into contact with the top frame of the MINY.
- ▶ Then remove the back wheels. To do this, raise the MINY slightly so that the back wheel to be removed is no longer in contact with the ground.
- ▶ Grasp the spokes on the back wheel and use your thumb to press the button located at the center of the wheel axle (Fig. 17).
- ▶ Keep the button pressed down and pull the wheel out of the axle receptacle. Make sure that the brake has been deactivated on the wheel being removed!
- ▶ Repeat the same steps with the second back wheel.
- ▶ The MINY is now ready for transport.

7.5 Unfolding the MINY

In order to unfold the MINY, proceed as follows:

- ▶ First, check to ensure that the MINY parking brake is deactivated.
- ▶ Pick the MINY up by the back end.
- ▶ Grasp the spokes on one of the back wheels and use your thumb to press the button located on the wheel axle.
- ▶ Keep the button pressed down and slide the wheel back into the axle receptacle on the frame.
- ▶ Release the button and check to ensure that the wheel is secure. Make sure that the wheel can no longer be pulled off!
- ▶ Attention: Do not press the button while performing this check.
- ▶ Repeat the same steps with the other back wheel.
- ▶ Fold the push bar unit backward until it audibly clicks into place.
- ▶ Then secure the pivoting top piece of the push bar into position, as described above (see Section 7.3.2).
- ▶ Then adjust the push bar to the desired height. To do this, pull the push bar to the desired height.
- ▶ Secure the push bar in position by tightening the quick-release connectors, as described above (see Section 7.3.2).
- ▶ If the clamping pressure is not sufficient (push bar still loose), then proceed as described above (see Section 7.3.2).
- ▶ Mount the seat shell

7.6 Drum brake-enabled back wheels

The MINY is equipped with two parking drum brakes on the two back wheels, which can be activated independently of each other. The brake levers are equipped with a three-level ratchet mechanism, which provides for varying levels of braking action.

7.6.1 Activating the parking brake

In order to activate the parking brake, proceed as follows:

- ▶ Engage both brake handles on the push bar (Fig. 18). Make sure that the brake handle does not return to the starting position (locking mechanism will click audibly into place); otherwise the parking brake has not been activated.

- ▶ Check to make sure that the brake is indeed activated by attempting to push the MINY.
- ▶ **Attention:** If the braking action is not sufficient, then have the brakes re-adjusted by your medical supplies dealer or readjust them yourself, as described in Section 8.2.
- ▶ You can also use the parking brakes for better control of the wheelchair on descents. In such cases, engage the parking brake, as needed, by pressing it into the first or second ratchet position only.

7.6.2 Deactivating the parking brake

In order to deactivate the parking brake (Fig. 19), proceed as follows: Deactivate the locking mechanism by activating the ratchet lever and allowing the brake lever to glide back into the starting position. You may need to release the ratchet lever further by pulling slightly on the brake lever before the brake can be deactivated.

8. Maintenance and care

The following section contains some instructions regarding the maintenance of the MINY seat unit wheelchair. For repairs and maintenance or if you have any questions about MINY, please always contact your specialist dealer.

After a repair the specialist dealer must conduct a performance test and confirm that the MINY is in a condition that is safe for use.

8.1 Checking the tyre pressure

MINY features pneumatic tyres as standard. In order to keep rolling resistance to an absolute minimum, you should check the tyre pressure at regular intervals. The tyres are standardly equipped with automobile valves, so you can check the pressure by using the tyre gauge at your local petrol station. The required filling pressure is printed on the tyres and is also specified in Section 6 ("Admissible tyre pressure") of these instructions.

To check the tyre pressure, please proceed as follows:

- ▶ Apply the parking brake (see section 7.8.1)
- ▶ Unscrew the valve cap from the valve by turning it anti-clockwise
- ▶ Press the tyre gauge nozzle firmly onto the valve
- ▶ Correct the tyre pressure to the specified value
- ▶ Remove the tyre gauge nozzle from the valve
- ▶ Screw the valve cap onto the thread of the valve (clockwise)
- ▶ Repeat the operation until you have checked all tyres
- ▶ If necessary, release the parking brake (see section 7.8.2)

Important: Depending on the version, the front and rear wheels require different tyre pressures.

8.2 Adjusting the parking brake

Tool: Size 8 spanner

Prerequisite: Insufficient braking power of the parking brake; MINY can be pushed even though the parking brake is applied.

To adjust the parking brake, proceed as follows:

- ▶ Make sure that the parking brake is not applied; if necessary, release the parking brake (see section 7.8.2)
- ▶ The Bowden cables of the parking brakes are routed through an adjusting bolt before the brake backing plates, which allows an adjustment of the braking power (Fig. 20)
- ▶ By turning this bolt anti-clockwise, the brake cable is brought under greater tension, pre-tensioning the brake. The adjusting bolt is secured by a nut
- ▶ Before adjusting the brake, release the securing nut by turning it to the left (depending on the version, a size 8 spanner may be required)

- ▶ Next, turn the adjusting bolt anti-clockwise until it becomes more difficult to turn the wheel. To check, lift the MINY on one side and turn the respective wheel with your other hand
- ▶ Then, turn the adjusting bolt clockwise until the wheel which has to be adjusted runs freely again
- ▶ Turn the counter nut located on the thread of the adjusting nut clockwise as far as it will go
- ▶ If necessary, repeat the procedure for the other wheel.

Advice: In order to check the adjustment of the brake, pull the brake lever several times before finally fastening the counter nut. If necessary, you have to repeat this operation, as some experience is required to carry out the adjustment.

8.3 Cleaning

MINY features a robust plastic coating. All parts can be cleaned with soap suds and a soft sponge. Avoid aggressive agents or abrasives as the surface of the plastic coating may be damaged. To disinfect MINY, any conventional disinfectant can be used.

Disinfection

Please use a water-based agent for disinfection, e.g. Terralin, Quartamon Med or Sagrotan Original concentrate. Please observe the manufacturer's instructions for use when doing so. It is best to clean plastic parts with a commercially available plastic cleaner (please observe the product information for the cleaning agent).

8.4 Service Intervals

MINY needs to be checked for functionality and defects by a specialist dealer or us regularly or according to usage, but at least once a year. The necessary checks are listed in the service plan on page 51 of this user manual.

8.5 Service life

Provided maintenance is performed regularly, the MINY is designed for a service life of 5 years. With appropriate care and maintenance the MINY will also be fit for use for a much longer period.

9. Transport /Transportation lock /Stair climbing

To transport the MINY undercarriage in handicapped transport vehicles, a certified and approved wheel chair restraint system should be used. For transport in cars, an additional 4-point occupant restraint system is needed. Please consider the separate manufacturer's user manual for correct fixing of the restraint systems.

Note: Never use the frame or the back of MINY for transport fastening!

Stair climbing: Overcoming stairs or curbs should be done with two people assisting. The assistants should grab the frame tubes and the rod on the back of the frame without lifting MINY.

The tilting mechanism has to be retracted and secured. When going up, the back assistant should pull MINY up while the front assistant stabilises the position by pushing both sides downward. When going down, the front assistant should decelerate MINY by pushing both sides downwards while the back assistant keeps the system in position by holding the back rod. To avoid damages of the wheels, move MINY slowly over the curbs.

10. Safety instructions

You are responsible for your child's safety. Your child's safety may be affected if you do not follow these instructions. Nevertheless, not all conditions that could possibly happen and all unpredictable situations can be covered. Common sense, caution and prudence are factors this product cannot possess; they are required from the person using this buggy.

The person using this wheelchair and the accompanying accessories should understand all instructions. He/she should explain how this buggy is operated to any person using this buggy and the accompanying accessories. Should the instructions be not clear and more precise explanations needed or should any further questions arise, contact your local specialised dealer or us.

- ▶ MINY is only authorised for intended usage.
- ▶ To avoid downfalls and dangerous situations, you should practise the use of your MINY on even surfaces and with the necessary assistance.
- ▶ Assisted by a second person, you should examine the effects on the performance of the MINY if the centre of gravity is shifted.

- ▶ The maximum load for MINY is 60 kg (incl. weight of user and seat shell incl. options).
- ▶ Seat units are only to be assembled to the MINY in the indicated directions of sight.
- ▶ The street undercarriage MINY is suitable for the interco seat units MAGICLINE, MAGICLIGHT, ORTHOLIGHT and ERGOLINE.
- ▶ Take care to always make sure that the seat unit is closely attached to the undercarriage and secured by the locking unit. Take care to always make sure that the locking of the trapezoidal adaptor is engaged and that the seat unit cannot be removed from the undercarriage.
- ▶ Take care to always make sure that the two spring-loaded catch mechanisms of the back recline are engaged if the MINY is used so that it may not fold up unintentionally.
- ▶ Always make sure the tilt stop system is in the extended position and secured by spring-loaded catch mechanisms so that it cannot retract accidentally.
- ▶ Before using the version with floating axles, take care to make sure that the floating axles are correctly engaged. To change the wheel base, always consult your specialist dealer or interco.
- ▶ When lifting MINY, only touch the indicated parts to avoid any injuries (see Section 9).
- ▶ Do not touch the movable parts when adjusting something. Otherwise your fingers might be squashed or become trapped.
- ▶ When finished with all adjustments, take care to lock all joints released before.
- ▶ Take care to regularly check all joints of MINY.
- ▶ If necessary, consult your specialised dealer.
- ▶ Should you have any questions concerning MINY, always contact your specialised dealer or interco.
- ▶ Do not use the footplate to get in or out of a seating system. The footplate should be folded down or removed beforehand.
- ▶ Secure the MINY by applying the brakes before getting in or out.
- ▶ Secure the patient against falling out before adjusting the seat tilt
- ▶ Never leave the patient unattended; not even when the patient is secured by a seat belt and all brakes are locked.
- ▶ Should the patient lean out too far or move too violently, there may be a danger of tipping over.
- ▶ When the MINY with seat unit is static, the parking brakes should always be applied.
- ▶ Do not carry the MINY when the seating system is attached and a patient seated within!

- ▶ Seat shell and push bars should not be used for carrying.
- ▶ Take care to clean all frame and plastic parts with mild cleaning agents only.
- ▶ Do not hang any shopping bag or similar things to the handle. This heightens the danger of tipping over.

Please take note of the following safety instructions:

The attachment of additional parts or alterations to MINY which have not been carried out by the manufacturer are forbidden and may be legally prosecuted should they become known. Please also take care that the MINY is only equipped with accessories supplied or approved of by the manufacturer. In case of non-compliance, the warranty of interco GmbH expires and no further liability claims against the manufacturer maintain.

11. Possible disturbances

Should any malfunctions appear (e. g. with the brakes, gas spring, seat unit locking etc.), please contact your local specialised dealer or us. Only specialist dealers authorized by interco are allowed to perform repairs.

12. Reuse and disposal

Reuse

MINY is suitable for reuse. Before it is passed on, all the necessary cleaning and maintenance activities must be performed. When it is handed over, make sure it is accompanied by all the documentation necessary for use (including these instructions for use).

Disposal

When MINY has reached the end of its service life, make inquiries with your local waste management company about the regulations concerning disposal and recycling.

13. Declaration of Conformity

EU Declaration of Conformity for Medical Products

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Manufacturer's address: Im Auel 50
53783 Eitorf
Germany

We hereby declare that the product

Name of the product: MINY® size 30-40 (art.-no. 77010-77011)
Design of the product: Undercarriage for seating systems with
tilt-in-space mechanism
Commission: Serial production with options
according to definition

complies with the relevant regulations of the following directive:

- ▶ Directive 93/42/EWG on medical products, annex I and VII

Eitorf, 02 May 2012

Michael Markwald
Chief Executive

14. Warranty

In addition to our General Terms and Conditions, we grant a warranty period of 1 year on the MINY products we supply. Within the warranty period, defective or faulty parts will be repaired or replaced free of charge. This warranty does not, however, cover modifications made during the course of product development.

We accept no liability for damage caused to our products in conjunction with products of any kind manufactured by other companies unless these have been expressly approved by us. Under certain circumstances, such combinations can represent a considerable danger.

Faults owing to natural wear or excessive straining as well as damage caused by excessive force and improper use are also excluded from this warranty. The warranty expires if parts other than genuine interco replacement parts have been used to carry out repairs. Certain bolts, nuts and spokes may become loose as a result of constant use. Please tighten these periodically, or ask your specialist dealer to do so for you.

Subject to technical alterations without prior notification

15. Service manual

Pos.	Area	Check	Adjustment / functionality	ok	Damage/ deformation	Exchange/ replacement
1	Frame	Frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Tilting support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
2	Trapezoidal adaptor		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
3	Steering wheel	Tires	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Tire pressure	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Running performance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Steering wheel fork	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Suspension on frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
4	Driving wheel	Tires	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Tire pressure	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Running performance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Axle shaft	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Drum brake	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
5	Brakes	Functional test 1	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Functional test 2	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Functional test 3	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
		Bowden wire	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	
6	Adjustment range	Gas springs	Damages	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
			Bolt connection	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		tilt-in-space mechanism	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Folding mechanism	Folding mechanism	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
			Arrester	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Back angle adjustment	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Push bar	Height adjustment	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
			Arrester	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Pos.	Area	Check			
		Adjustment / functionality	ok	Damage/ deformation	Exchange/ replacement
7	Bolt connections		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Cover flaps in place	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
8	Options	Wheel guard	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Spoke guard	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Middle foot plate	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
9	Optics (Visual inspection)				
		Paint	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Cleanliness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Type label	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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Status as per March 2018
Subject to technical alterations
without prior notification

Zertifiziert nach
Certified as per
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 357-1
DIN EN ISO 13485:2012
Reg.-Nr. 70 105 357-1
TÜV Hessen

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© interco GmbH, Eitorf, 2018