ROOMY NEW EDITION®

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Contents

1.	Introduction	22
2.	General	22
3.	Intended Use/Application	23
4.	Equipment/Options	23
5.	Technical Data	24
6.	Use/Assembly	25
6.1	Sliding handle attachment	25
6.2	Attachment of the seat shell	26
6.3	Removal of the seat shell	27
6.4	Adjustment of the seat inclination	28
6.5	Adjustment of the seat height	29
6.6	Attachment of a footrest	30
6.7	Adjustment of the sliding handle angle (optional)	30
6.8	Brakes	31
7.	Care/Cleaning	32
8.	Safety instructions	32
9.	Reuse and disposal	34
10.	Declaration of conformity	35
11.	Warranty	35
12.	Service plan	36

1. Introduction

Congratulations, you have chosen the ROOMY NEW EDITION seat unit frame by interco GmbH.

In order to ease the handling of the product for you, we described the application of the different components in the following user manual. Please be appreciative of the fact that we also describes parts that might not be found in on your product. Most important is the individual fitting to the patient's needs. In order to satisfy most claims, the list of described items is a little more comprehensive. Please read the parts that apply to your specific model carefully and keep the user manual for later consulting.

For further questions please contact us or your specialised dealer.

2. General

ROOMY NEW EDITION is a mobile indoor undercarriage which is custom made to meet the user's requirements indoors. The entire construction consists of oval steel tubing which is permanently protected with environmentally friendly plastic coating.

ROOMY NEW EDITION is available in several versions:

With four swivelling 100 mm or 125 mm plastic castors which can be braked and their direction locked. Additionally, two versions with 12" or 16" drum-braked air-filled or accident-secure wheels are available. They are equipped with a sliding handle and a parking brake and a Bowden wire. In order to enable driving around, the 125 mm and 100 mm castor version of the ROOMY NEW EDITION ought to be equipped with an additional extra removable sliding handle. Because ROOMY NEW EDITION must adapt to the needs of very young children and older children, emphasis was placed on the provision of attachment possibilities for all our seat shell types and sizes when it was designed. The continuously variable height adjustment of the attached seat shell guarantees safe use at home and in institutions such as the kindergarten. Integration in the kindergarten group is now no longer a problem as the seat shell can be lowered almost to the floor. Participation in the daily eating ritual is now easy because the undercarriage can be raised up to the height of the table. Our seat shell systems MAGICLIGHT, MAGICLINE, ORTHOLIGHT and ERGOLINE Passiv can be combined with ROOMY NEW EDITION using our trapezoidal adapter system. The seat system MAGICLINE

which grows with its users also fits in all sizes. Additionally, the multifunctional adapter plate fits all commonplace seat shell types supplied by other manufacturers. We as interco naturally give you the guarantee for the procurement of spare parts or new adaptation for several years to come.

3. Intended Use /Application

When supporting people with seat shells and modular seat systems, chassis and undercarriages are necessary, both for indoors and outdoors. They facilitate the mobility and transport of handicapped persons. The undercarriage ROOMY NEW EDITION is a seat shell undercarriage which can be used both outdoors and indoors, dependent on its outfit. The ROOMY NEW EDITION is specially developed for the accommodation of orthopaedic seat systems by the company interco GmbH and can be adjusted and set to multiple positions, thus enabling optimum individual adaptation. It enables easy lowering a patient in the seat system to a very low seat height and lifting to a height for nursing. Appropriate settings must be made to adapt to the respective seat system and should only be carried out by orthopaedic or rehabilitation specialist shops.

4. Equipment /Options

ROOMY NEW EDITION is available in several versions:

1. With 100mm or 125 mm plastic castors without a sliding handle, only for indoor use.
2. With 16" wheels and a sliding handle for indoors and outdoors, accident-secure or with pneumatic tyres.
3. Or with 12" wheels and a sliding handle for indoors and outdoors, accident-secure or with pneumatic tyres.

Optional Equipment

The ROOMY NEW EDITION with 100mm/125 mm castors can be equipped with an adjustable sliding handle (Photo 1).

This is already included in the scope of delivery for the ROOMY NEW EDITION with 12" or 16" wheels.

5. Technical Data

	with 100 mm / 125 mm castors	with 12" wheels	with 16" wheels
Weight without the patient	approx. 16 kg / 17 kg + approx. 2 kg sliding handle	approx. 21 kg + approx. 2 kg sliding handle	approx. 22 kg + approx. 2 kg sliding handle
Maximum load	70 kg incl. seat shell	70 kg incl. seat shell	70 kg incl. seat shell
Maximum lenght	850 mm / 900 mm excl. sliding handle	990 mm excl. sliding handle	1040 mm excl. sliding handle
Total width	580 mm / 600 mm	700 mm	700 mm
Adapter plate angle adjustment	0° to +35° -10° to +25° (optional)	0° to +35° -10° to +25° (optional)	0° to +35° -10° to +25° (optional)
Maximum adapter plate total height to the top edge	650 mm / 680 mm	750 mm	750 mm
Minimum adapter plate total height to the top edge	210 mm / 240 mm	315 mm	315 mm
Tyre pressure if pneumatic tyres are used at the front	-	2.5 bar max.	2.5 bar max.
Tyre pressure if pneumatic tyres are used at the back	-	2.5 bar max.	2.5 bar max.


Please observe that you must reach a compromise between the patient's roller comfort (low air pressure) and pushing comfort for the accompanying person (high air pressure). Find this by trial and error. The factory setting is made at 2.5 bar.

6. Use / Assembly

ROOMY NEW EDITION is supplied in this model version with pre-mounted 100 mm and 125 mm castors. You can use your ROOMY NEW EDITION immediately after removal of the transport packaging. The sliding handle must be mounted extra for the model versions with 12 "and 16 " wheels.



6.1 Sliding handle attachment

The acceptance for the sliding handle is found at the rear end of the undercarriage. You can mount it at three different positions (Photo 2) by means of three catches and the relevant bores. Firstly, pull out the pull latches at the relevant handle and turn them by 90°. Insert the handle from the top at the angular position desired into the acceptance (Photo 3). Press the handle into the acceptance. You can feel a distinct engagement when the handle is firm in the acceptance (Photo 4). Then turn the pull latches back to their initial positions until they get engaged, to lock the handle (Photo 5).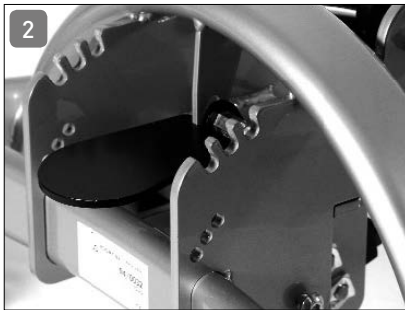

6.2 Attachment of the seat shell

Fix the seat shell to the undercarriage adapter plate (Photo 6) using the trap-ezoidal adapter. For this purpose, carry out the following work steps and check whether the seat shell is securely held after attaching it to the adapter plate. First of all, unlock the seat shell fixing by pulling the wire catch out and rotating it 90° (Photo 7). Place the back end of the seat shell adapter on the front side of the adapter plate so that the catch is pressed downwards (Photo 8). Then press the seat shell back until it engages and this can be clearly heard (Photo 9). Re-lock the seat shell fixing by turning the wire catch 90° again until it engages (Photo 10). Finally, check whether the seat shell locking is secure (Photo 11).

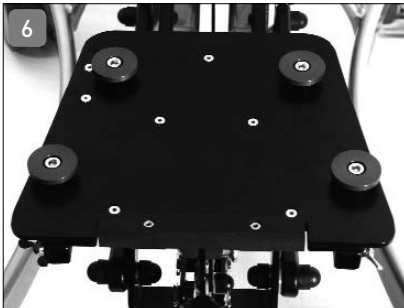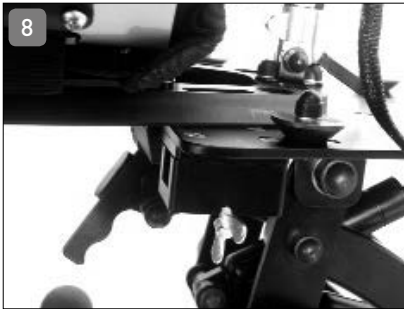
6.3 Removal of the seat shell

Firstly pull the wire catch out and turn it 90° (Photo 12). Open the seat shell locking by pressing the handle upwards (Photo 13). Now it is possible to pull the seat shell out forwards (Photo 14).


6.4 Adjustment of the seat inclination

The adapter plate and therefore the seat shell can be adjusted continuously by means of a gas pressure spring as to its inclination from 0° to +35° (optionally -10° to +25°) (Photos 15 and 16). Loosen the catch for the angle of inclination first. For this purpose, pull the wire catch out and rotate it 90° (Photo 17). Then press the adjusting lever and incline the seat shell or the adapter plate to the desired position (Photos 18 and 19). Let the handle slide back upwards. Turn the wire catch again until it clicks into place in order to stop the adjusting lever (Photo 17).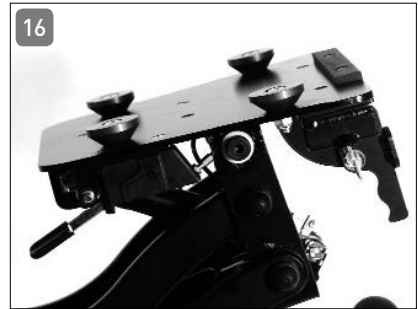

6.5 Adjustment of the seat height

The adapter plate and consequently the seat shell can be adjusted continuously variably using a gas pressure spring at a height between 200mm and 650mm. (ROOMY NE with 100mm/125 mm plastic castors) or 300mm to 750mm (ROOMY NE with 12" or 16" wheels) (Photos 20 and 21). The seat height is adjusted using the foot pedal situated at the rear end of the ROOMY NEW EDITION (Photo 22). Firstly release the height adjustment by pulling out the wire catch and rotating it 90° (Photo 23). Then operate the foot pedal and lift or lower the seat area manually to the desired height (Photo 24). Turn the wire catch again until it clicks into place.

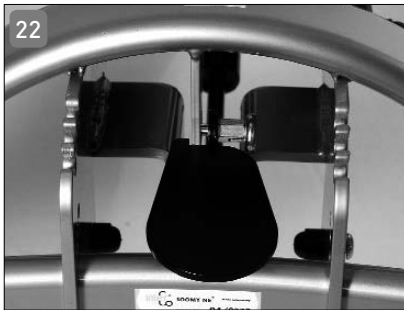6.6 Attachment of a footrest

The ROOMY NEW EDITION also has an attachment for footrests from the interco range. In order to attach a footrest, first remove the wing screws for the footrest attachments on the front lefthand and right-hand side of the adapter plate (Photo 25). Press the footrest holder into the attachment and re-tighten the wing screws.

6.7 Adjustment of the sliding handle angle (optional)

To adjust the inclination angle of the handle first loosen the pull latches on both sides at the lower end of the sliding handle by a 90° turning (Photo 26). The undercarriage is provided with three catches and bores to mount the sliding handle (Photo 27). Mount the sliding handle at the position desired of the three to define the inclination angle. For that proceed as described in the section Attachment sliding handle (Chapter 6.1).

6.8 Brakes

Locking the wheels – ROOMY NE with 100 mm or 125 mm plastic castors

In order to lock the wheels, apply the locking pedals for the castors using your foot (Photo 28). In order to release the brakes again, pull the pedal up again using the tip of your foot.

Locking the wheels – ROOMY NEW EDITION with 12" or 16" wheels (Photo 29)


The 12" and 16" ROOMY NEW EDITION sets of tyres only differ with regard to size and visual appearance. However, the operation of the locking brake (drum brake) remains identical.

The levers for the locking brake are situated on the sliding handle (Photo 30). In order to block the wheels, pull the brake lever (Photo 31). The red locking lever engages three times of its own accord. In order to re-release the brake, apply the black brake lever and the red brake lever together (Photo 32). The brake can consequently also be used as a running brake, for example to reduce downhill speed.

Setting the brake force for a driving wheel with a drum brake at ROOMY NEW EDITION with 12" or 16" wheels

In order to achieve an optimum braking effect, the brake force is adjusted using the setting screw (Photo 33/1). The brake power is intensified by unscrewing the setting screw and decreased by screwing it in. Please unscrew the screw until you hear grinding noises on the rotating wheel. Then screw the setting screw back in until the grinding noises disappear. Now the wheel runs smoothly. After completion of settings, the setting screw is fixed by tightening the locknut (Photo 33/2). Pay attention that the brake force is set to the same level for both driving wheels.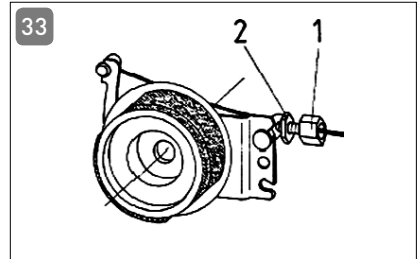

Correct setting of the drum brakes:

The braking effect must be adjusted so that the wheelchair remains standing on a 7° slope in the third brake catch with a load of 70 kg. When carrying out assembly and adjusting, there is a risk of bruising one's limbs. Never grasp moving parts whilst carrying out this work! Always make sure that all previously loosened connections are re-tightened correctly!

7. Care /Cleaning

ROOMY NEW EDITION is provided with a high-quality, resistant plastic coating.

Clean ROOMY NEW EDITION with soap suds and a soft sponge. Avoid aggressive or abrasive cleaning agents as these may damage the surface. All normal disinfectants for this area of use may be used.

8. Safety instructions

- ▶ ROOMY NEW EDITION is only authorised for its intended use.
- ▶ In order to avoid falling out and dangerous situations, you should practice dealing with your new ROOMY NEW EDITION initially.
- ▶ ROOMY NEW EDITION may only be used on even, solid and horizontal ground.

- ▶ Discover the effects of changes to the centre of gravity on the reaction by the ROOMY NEW EDITION whilst securely supported by a helper.
- ▶ Seat units may only be mounted on the ROOMY NEW EDITION in the stipulated direction (the direction of travel).
- ▶ Always pay attention that the seat shell is attached securely to the undercarriage and secured using the pull snatch.
- ▶ In order to avoid injury, only use the given operating areas. Do not interfere with the moving parts during setting processes.
- ▶ After completion of settings, close the previously opened connections securely again.
- ▶ The maximum load for ROOMY NEW EDITION is 70 kg (seat system + patient + accessories).
- ▶ The attachment of additional parts or changes to the ROOMY NEW EDITION by persons other than the manufacturer is not permitted and may be legally punished if recognised. This also leads to the expiry of the duty of guarantee by the company interco GmbH and no liability rights from the manufacturer exist in this case.
- ▶ Hanging bags onto it is not permitted.
- ▶ Subject all ROOMY NEW EDITION connections to regular visual checks. If you have any doubts, always refer to your specialist dealer.
- ▶ Always refer to your specialist dealer or to interco if you have any questions regarding the ROOMY NEW EDITION.
- ▶ Do not use the footrest when getting into or out of a seat system. It should be folded away or removed previously.
- ▶ Secure ROOMY NEW EDITION by applying the brakes before all lifting and dropping processes and when getting in and out.
- ▶ Secure the patient against falling out before all lifting and dropping processes and before altering the seat inclination.
- ▶ Never leave the patient unattended, not even if he is secured by a belt and the brakes are applied.
- ▶ If the patient leans out a long way or makes violent movements, it is possible that it will tip over.
- ▶ Only park ROOMY NEW EDITION with a seat unit if the locking brakes are engaged.
- ▶ Only push the indoor undercarriage when the seat system is lowered (tilting risk).
- ▶ In order to manage stairs, the patient, the seat system and the indoor undercarriage must be transported separately from each other. Carrying the ROOMY NEW EDITION with an attached seat system and a patient sitting on it is not permitted!

- ▶ It is a prerequisite for transporting the ROOMY NEW EDITION care system in transport vehicles for the handicapped (BTWs) and motor vehicles that the respective transport contractor has an examined and registered wheelchair support system.
- ▶ All frames and plastic parts may only be cleaned using mild cleaning agents.

9. Reuse and disposal

Reuse

ROOMY NEW EDITION is suitable for reuse. Before it is passed on, all the necessary cleaning and maintenance activities must be performed. When it is handed over, make sure it is accompanied by all the documentation necessary for use (including these instructions for use).

Disposal

When ROOMY NEW EDITION has reached the end of its service life, make inquiries with your local waste management company about the regulations concerning disposal and recycling.

10. Declaration of conformity

EU-Declaration of Conformity for Medical Devices

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Address of the manufacturer: Im Auel 50
53783 Eitorf GERMANY

We hereby declare that the device

Article description: ROOMY NEW EDITION - indoor undercarriage
Type: Art. no. 49000, 49100, 49200

conforms to the pertinent provisions of the directive listed below:

- ▶ Medical Device Directive 93/42/EEC, Annex I and VII

Eitorf, 05 February 2004

Michael Markwald
CEO

11. Warranty

As a supplement to the General Standard Terms and Conditions, we guarantee the product ROOMY NEW EDITION provided by us for 1 year. After you send in the guarantee card, we give you an additional year's manufacturer's guarantee, thus creating a total guarantee period of 2 years.

Unusable or faulty parts shall be repaired or replaced free of charge during the guarantee period. Development-related changes and worn parts are excluded from this ruling (gas pressure dampers, wheels and tyres). We cannot take on any liability for damages caused by combining our products with any kind of third party products which can include considerable dangers themselves. Only prod-

ucts from other manufacturers which are expressly approved by our company are an exception to this ruling.

Furthermore, faults which are based on natural wear and tear, excessive use, violent and deliberate damages and non-determined use are excluded from this ruling.

Subjekt to technical changes.

12. Service plan

Pos.	Area	Check	Check		
			ok	Damage / deformation	Exchange / replacement
1	Main frame	Main frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
2	Trapezoidal adaptor		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3	Steering wheel	Suspension on frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Running performance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4	Brakes	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
5	Adjustment range				
	Gas springs	Damages	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Bolt connection	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Seat adjustment	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Push bar	Height adjustment	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Arrester	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
6	Bolt connections	Size and length	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Completeness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
7	Options	Push bar	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Foot rest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
8	Optics (Visual inspection)				
	Paint		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Cleanliness		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Type label		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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Status as per March 2018
Subject to technical alterations
without prior notification

Zertifiziert nach
Certified as per
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 357-1
DIN EN ISO 13485:2012
Reg.-Nr. 70 105 357-1
TÜV Hessen

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© interco GmbH, Eitorf, 2018