ROOMY®
ROOMY®

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Contents

1.	Introduction	16
2.	General information	16
3.	Intended Use/Application	16
4.	Equipment / Options	17
5.	Technical Data	17
6.	Operation and Assembly	18
6.1	Attachment of the seat unit	18
6.2	Adjustment of the seat height	18
6.3	Adjustment of the seat inclination	18
7.	Maintenance/Care	19
7.1	Cleaning	19
7.2	Service Intervals	19
8.	Safety Instructions	19
9.	Possible disturbances	21
10.	Reuse and disposal	21
11.	Declaration of conformity	22
12.	Warranty	23
13.	Service plan	24

1. Introduction

Congratulations, you have chosen the ROOMY seat unit frame by interco GmbH. In order to ease the handling of the product for you, we described the application of the different components in the following user manual. Please be appreciative of the fact that we also describe parts that might not be found in on your product. Most important is the individual fitting to the patient's needs. Please read the parts that apply to your specific model carefully and keep the user manual for later consulting.

ROOMY is the appropriate seat unit underframe for the complete interco seat shell program and it offers several useful functions. It can be adjusted in height via gas pressure springs and in inclination angle via locking pins; four swivelling 75 mm plastic castors are steerable and individually lockable. Hence, Roomy is very agile and can be used ideally in narrow rooms.

2. General information

Colour differences between individual parts cannot always be avoided. Unfortunately, any claims referring to this fact cannot be admitted. Please understand that improper handling, possible waterline stains or differences in the material colours, as described above, are no basis for a justified claim. A regular maintenance of all metal parts of the chair, in particular the chrome parts, is necessary to preserve its high-grade appearance. Conventional cleaning products are sufficient for this purpose. Small repair sets for damages to the frame's paintwork are available at a specialised dealer.

3. Intended Use / Application

When supplying children with seat units and modular seating systems, undercarriages for indoor and/or outdoor use are compulsory. They increase the mobility and make the transport of the insured person possible. Therefore, the purpose of all undercarriages – in particular ROOMY – always is the reception of a seat unit.

4. Equipment / Options

ROOMY's basic features are:

- ▶ Trapezoidal seat unit adaptor for interco seat shells
- ▶ 75 mm swivel wheels
- ▶ Manual height adjustment via gas pressure spring
- ▶ Manual seat inclination via locking pins

Optional Equipment:

- ▶ Adjustable handle in short edition:
Push the push bar adjusting unit underneath the trapezoidal adapter plate and fasten both units with screws and self-securing nuts. Push the push bar into the adapter tube of the adjusting unit and fasten the wing screws. The push bar is equipped with a simple spring clip to avoid that the push bar can be pulled out completely.

Note: Make sure that all screws and clamps are correct and tight.

- ▶ Continuous/split foot rest with knee angle:
In order to adjust a foot rest, loosen the wing screws of the foot rest adapter on the left and right front part of the adapter plate. Push the foot rest holder into the adapter and fasten the screws.

5. Technical Data

Weight without seat shell	approx. 12 kg
Max. Load	60 kg incl. seat shell and patient
Max. length	590 mm
Total width	500 mm
Inclination adjustment adapter plate	-10° to +30°
Max. total height (to upper edge of adapter plate)	approx. 500 mm
Min. total height (to upper edge of adapter plate)	approx. 370 mm

6. Operation and Assembly

6.1 Attachment of the seat unit

The seat unit is fixed in the trapezoid adapter plate with the help of the trapezoid adapter. Carry out the following steps and check the safe halt of the seat unit afterwards. Unlock the seat shell fixation by pulling the locking pin and turning it 90°. Position the seat unit with the rear part of the trapezoidal adapter on the trapezoidal plate and push the seat unit (and thereby the trapezoidal adapter) backwards into the trapezoid plate until the locking pin engages audibly. Lock the seat shell fixation again by turning the locking pin once more by 90°. At last, check that the seat shell and the lock are firmly fixed.

Removal of the seat unit

Pull the locking pin downward and turn by 90°. Open the seat shell fixation by pushing down the lever. Now you can pull the seat shell out towards the front.

6.2 Adjustment of the seat height

The adapter plate and consequently the seat shell can be adjusted continuously variably using a gas pressure spring at a height between 370 mm and 500 mm (ROOMY with 75mm plastic castors).

To adjust the height of the seat (adapter plate) please approach as follows:

- ▶ Push down the adapter plate with one hand. At the same time grab underneath the trapezoidal adapter plate and pull the gas pressure handle upwards.
- ▶ The gas pressure spring automatically pushes the seat upward when you release the pressure onto the trapezoidal adapter plate. When increasing the pressure, the height of the trapezoidal adapter can be lowered.
- ▶ Release the lever when the seat is in the desired position.

6.3 Adjustment of the seat inclination

The adapter plate and therefore the seat shell can be adjusted continuously by means of a locking pin.

To adjust the seat inclination, please approach as follows:

- ▶ Hold the seat shell at the back rest to avoid a sudden tilting of the seat shell.
- ▶ Pull the locking pin and at the same time push the seat shell upward or downward until the desired inclination angle is reached.
- ▶ Release the locking pin until it catches into the appropriate bore hole.

7. Maintenance / Care

The following chapter gives some directions for maintenance and care of the ROOMY seat shell underframe. For comprehensive repairs or for additional questions about ROOMY, please approach your specialised dealer.

7.1 Cleaning

ROOMY features a robust plastic coating. All parts can be cleaned with soap suds and a soft sponge. Avoid aggressive agents or abrasives as the surface of the plastic coating may be damaged. To disinfect, any conventional disinfectant can be used.

7.2 Service Intervals

ROOMY needs to be checked for functionality and defects by a specialist dealer or us regularly or according to usage, but at least once a year.

8. Safety Instructions

You are responsible for your child's safety. Your child's safety may be affected if you do not follow these instructions. Nevertheless, not all conditions that could possibly happen and all unpredictable situations can be covered. Common sense, caution and prudence are factors this product cannot possess; they are required from the person using this buggy. The person using this buggy and the accompanying accessories should understand all instructions. He/she should explain how this buggy is operated to any person using this buggy and the accompanying accessories. Should the instructions be not clear and more precise explanations needed or should any further questions arise, contact your local specialised dealer or us.

- ▶ ROOMY is only authorised for its intended use.
- ▶ In order to avoid falling out and dangerous situations, you should practice dealing with your new ROOMY initially.
- ▶ ROOMY may only be used on even, solid and horizontal ground.
- ▶ Discover the effects of changes to the centre of gravity on the reaction by the ROOMY whilst securely supported by a helper.
- ▶ Seat units may only be mounted on the ROOMY in the stipulated direction (the direction of travel).
- ▶ Always pay attention that the seat shell is attached securely to the under-carriage and secured using the pull snatch.

- ▶ In order to avoid injury, only use the given operating areas. Do not interfere with the moving parts during setting processes.
- ▶ After completion of settings, close the previously opened connections securely again.
- ▶ The maximum load for ROOMY is 60 kg (seat system + patient + accessories).
- ▶ Hanging bags onto it is not permitted.
- ▶ Subject all ROOMY connections to regular visual checks. If you have any doubts, always refer to your specialist dealer.
- ▶ Always refer to your specialist dealer or to interco if you have any questions regarding the ROOMY.
- ▶ Do not use the footrest when getting into or out of a seat system. It should be folded away or removed previously.
- ▶ Secure ROOMY by applying the brakes before all lifting and dropping processes and when getting in and out.
- ▶ Secure the patient against falling out before all lifting and dropping processes and before altering the seat inclination.
- ▶ Never leave the patient unattended, not even if he is secured by a belt and the brakes are applied.
- ▶ If the patient leans out a long way or makes violent movements, it is possible that it will tip over.
- ▶ Only park ROOMY with a seat unit if the locking brakes are engaged.
- ▶ Only push the indoor undercarriage when the seat system is lowered (tilting risk).
- ▶ In order to manage stairs, the patient, the seat system and the indoor undercarriage must be transported separately from each other. Carrying the ROOMY with an attached seat system and a patient sitting on it is not permitted!
- ▶ All frames and plastic parts may only be cleaned using mild cleaning agents

Please also note the following safety instructions:

The attachment of additional parts or changes to the ROOMY by persons other than the manufacturer is not permitted and may be legally punished if recognised. Please also note that only accessories provided or approved by the manufacturer may be used with ROOMY. Failure to comply leads to the expiry of the duty of guarantee by the company interco GmbH and no liability rights from the manufacturer exist in this case.

9. Possible disturbances

Should any malfunctions appear (e.g. with the brakes, gas spring, seat unit locking etc.), please contact your local specialised dealer or us.

10. Reuse and disposal

Reuse

ROOMY is suitable for reuse. Before it is passed on, all the necessary cleaning and maintenance activities must be performed. When it is handed over, make sure it is accompanied by all the documentation necessary for use (including these instructions for use).

Disposal

When ROOMY has reached the end of its service life, make inquiries with your local waste management company about the regulations concerning disposal and recycling.

11. Declaration of conformity

EU-Declaration of Conformity for Medical Devices

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Address of the manufacturer: Im Auel 50
53783 Eitorf GERMANY

We hereby declare that the device

Article description: ROOMY
Type: Art. no. 48011

conforms to the pertinent provisions of the directive listed below:

- ▶ Medical Device Directive 93/42/EEC, Annex I and VII

Eitorf, 22 February 2010

Michael Markwald
CEO

12. Warranty

As a supplement to the General Standard Terms and Conditions, we guarantee the product ROOMY provided by us for 1 year. After you send in the guarantee card, we give you an additional year's manufacturer's guarantee, thus creating a total guarantee period of 2 years.

Unusable or faulty parts shall be repaired or replaced free of charge during the guarantee period. Developmentrelated changes and worn parts are excluded from this ruling (gas pressure dampers, wheels and tyres). We cannot take on any liability for damages caused by combining our products with any kind of third party products which can include considerable dangers themselves. Only products from other manufacturers which are expressly approved by our company are an exception to this ruling.

Furthermore, faults which are based on natural wear and tear, excessive use, violent and deliberate damages and non-determined use are excluded from this ruling.

Subjekt to technical changes.

13. Service plan

Pos.	Area	Check	ok	Damage / deformation	Exchange / replacement
1	Main frame	Main frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
2	Trapezoidal adaptor		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3	Steering wheel	Suspension on frame	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Running performance	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4	Brakes	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
5	Adjustment range				
	Gas springs	Damages	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Bolt connection	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Seat adjustment	Functional test	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Push bar	Height adjustment	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Arrester	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
6	Bolt connections	Size and length	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Completeness	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
7	Options	Push bar	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
		Foot rest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
8	Optics (Visual inspection)				
	Paint		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Cleanliness		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
	Type label		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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Status as per March 2018
Subject to technical alterations
without prior notification

Zertifiziert nach
Certified as per
DIN EN ISO 9001:2015
Reg.-Nr. 73 100 357-1
DIN EN ISO 13485:2012
Reg.-Nr. 70 105 357-1
TÜV Hessen

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© interco GmbH, Eitorf, 2018